

QUALITY PERFORMANCE IN A LATEX-FREE GLOVE CLEARED FOR USE WITH CHEMOTHERAPY

THE WORRY-FREE CHOICE...DESIGNED FOR USE BY EVERYONE, EVERYWHERE

HALYARD* STERLING* Nitrile Exam Gloves provide excellent fit and feel compared to latex. The new nitrile formulation provides excellent comfort and stretch compared to the previous glove, without sacrificing any of STERLING* Nitrile's barrier properties or durability. They also have an Acceptable Quality Level (AQL) of 1.0. That means STERLING* Exam Gloves are significantly less likely to have critical defects than exam gloves that just meet the ASTM 2.5 AQL requirement.

If you want to standardize on one exam glove for virtually all your needs, facility-wide, STERLING* Nitrile is the glove to choose.

PERFORMANCE IN A LATEX-FREE EXAM GLOVE

Latex allergy is a concern within the healthcare industry. With STERLING* Nitrile Exam Gloves, Halyard Health has created a latex-free glove with the protection, comfort, and sensitivity of latex, at a comparable value.

- **The Protection of Nitrile**

- Halyard's manufacturing process boosts the tensile strength of nitrile—the force required to break through the glove—to increase its already superior barrier protection qualities.

- **The Tactile Sensitivity of Latex**


- The process also allows a reduction in glove thickness, which means that less force is required to stretch the material. That provides a latex-free glove with the dexterity and tactile sensitivity of natural rubber latex.

SAVE SPACE AND WASTE

Standard length STERLING* Nitrile Exam Gloves come in space saving 200 count dispenser boxes. That means up to a 50% saving in the storage space you'd usually devote to gloves. It also means fewer run-outs and replacements in use, which can help increase convenience and compliance. It also means there are fewer boxes to discard, which means less waste for your facility.

SMARTPULL* DISPENSER BOX

The SMARTPULL* dispensing incorporates 2 separate openings on the box. The first, smaller opening is used when the box is full and helps control dispensing to lessen waste from dropped gloves or multiple dispensing. When the box is half empty, the second, larger opening allows easier access to the gloves.

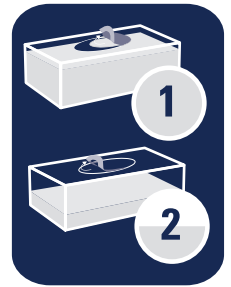

Tested Chemotherapy Drug and Concentration

Average Breakthrough Detection Time (minutes)

Cyclophosphamide (20.0 mg/ml)	No breakthrough up to 240 minutes
Doxorubicin HCl (2.00 mg/ml)	No breakthrough up to 240 minutes
Etoposide (20.0 mg/ml)	No breakthrough up to 240 minutes
5-Fluorouracil (50.0 mg/ml)	No breakthrough up to 240 minutes
Paclitaxel (Taxol) (6.0 mg/ml)	No breakthrough up to 240 minutes
Cisplatin (1.0 mg/ml)	No breakthrough up to 240 minutes
Dacarbazine (10.0 mg/ml)	No breakthrough up to 240 minutes
Ifosfamide (50.0 mg/ml)	No breakthrough up to 240 minutes
Mitoxantrone (2.0 mg/ml)	No breakthrough up to 240 minutes
Vincristine sulfate (1.0 mg/ml)	No breakthrough up to 240 minutes
Carmustine (3.3 mg/ml)	Not for use with Carmustine
ThioTEPA (10.0 mg/ml)	Not for use with ThioTEPA



200 Count Box Saves Space and Waste!

Product Specifications

Gauge Thickness Measurements	MM	MIL
Middle Finger:	.09	3.8
Palm:	.08	2.8
Cuff:	.07	2.4
Average Length	242mm	(9.5")
	Xtra [*] 310mm	(12.0")

Physical Properties

Before Aging

Tensile Strength:	42 MPa
Ultimate Elongation:	650%

After Aging

Tensile Strength:	38 MPa
Ultimate Elongation:	550%

Quality Standards

Exceeds current ASTM D 6319: Standard Specification for Nitrile Examination Gloves for Medical Application.

Biocompatible as measured by Primary Skin Irritation, Repeat Challenge Sensitization, and Hemolysis Testing.

Note: All specifications are subject to change without notice.

Other Benefits:

- Box opening – significantly reducing the incidence of getting more than one glove while dispensing
- Enhanced comfort and stretch
- STERLING[®] Nitrile Exam Gloves have an AQL of 1.0 for critical defects. This means that STERLING[®] Exam Gloves are significantly less likely to have critical defects than exam gloves that just meet the ASTM 2.5 AQL requirement



KNOWLEDGE NETWORK* Clinical Education
 Knowledgeable Customer Support
 Expert Sales Force
 Tools & Best Practices
 Clinical Research
 Commitment to Excellence

For more information, please visit:

www.halyardhealth.co.uk

EN The STERLING[®] Nitrile Powder-Free Exam Glove was tested with the front chemotherapy drug concentrations in accordance with ASTM D 6978, Standard Practice for Assessment of Resistance of Medical Gloves to Permeation by Chemotherapy Drugs.

CAUTION: The testing conditions used are intended to approximate the worse case conditions for clinical use. Testing was conducted on single layer glove material. It is the user's responsibility to determine the applicability of these gloves for the intended use with chemotherapy drugs.

Storage Recommendations: Store in a cool, dry place. Open box should be shielded from exposure to direct sunlight, intense artificial light, x-ray machines and other sources of ozone.

HALYARD[®] STERLING[®] Nitrile Exam Gloves



- Non-sterile
- Ambidextrous
- Textured fingertips
- Beaded cuffs
- Higher count, saves space
- Cleared for use in chemotherapy

Item	Description	Size	Packaging
44987	STERLING [®] Nitrile Powder-Free Exam Gloves	X-Small	200 eaches/box 10 boxes/case
44988	STERLING [®] Nitrile Powder-Free Exam Gloves	Small	200 eaches/box 10 boxes/case
44989	STERLING [®] Nitrile Powder-Free Exam Gloves	Medium	200 eaches/box 10 boxes/case
44990	STERLING [®] Nitrile Powder-Free Exam Gloves	Large	200 eaches/box 10 boxes/case
44991	STERLING [®] Nitrile Powder-Free Exam Gloves	X-Large	170 eaches/box 10 boxes/case

HALYARD[®] STERLING[®] Nitrile-XTRA Exam Gloves

- 310mm length for more protection against fluid and splash
- Extended, beaded cuff for better coverage over the wrist
- Certified to meet NFPA 1999:2008
- Cleared for use in chemotherapy

Item	Description	Size	Packaging
44286	STERLING [®] Nitrile-XTRA [®] Powder-Free Exam Gloves	X-Small	100 eaches/box 10 boxes/case
44287	STERLING [®] Nitrile-XTRA [®] Powder-Free Exam Gloves	Small	100 eaches/box 10 boxes/case
44288	STERLING [®] Nitrile-XTRA [®] Powder-Free Exam Gloves	Medium	100 eaches/box 10 boxes/case
44289	STERLING [®] Nitrile-XTRA [®] Powder-Free Exam Gloves	Large	100 eaches/box 10 boxes/case
44290	STERLING [®] Nitrile-XTRA [®] Powder-Free Exam Gloves	X-Large	100 eaches/box 10 boxes/case